Important

The PiKo is intended for single-patient use, as described in the user manual.

It can however be adapted for multi-patient use when fitted with Ferraris one-way mouthpieces (part number 600-30F).


When used in this way the PiKo requires a one-way mouthpiece **to be fitted for each patient, and discarded after each test.**

The maintenance described in the user manual applies to both modes of use but once used in multi-patient mode the PiKo becomes multi-patient only. The PiKo cannot be fully disinfected. Only Ferraris one-way mouthpieces (600-30F) have been validated for use with the PiKo.

Adapting the PiKo

When used in multi-patient mode, the plastic mouthpiece of the PiKo requires an adapter (part number 120340). This should be fitted to the PiKo and permanently kept with it from now on. It may be periodically washed as described in the maintenance section of the PiKo User Manual.

- A. Push the adapter ring half way onto the PiKo's mouthpiece (2). The adapter should be kept in this position.
- B. Hold the assembly as shown (3). Position the one-way mouthpiece over the end of the adapter; using a slight twisting motion push it until it reaches half way over the adapter. Remove in reverse. Always renew one-way mouthpieces between patients.

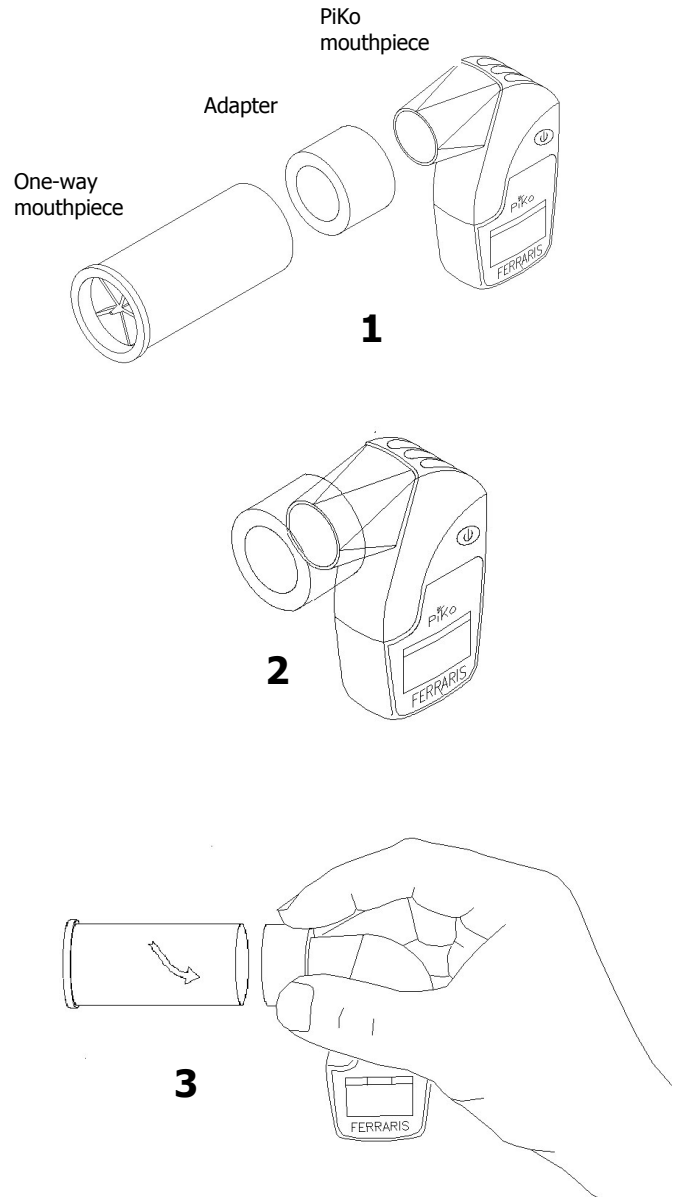- 1. One-way adapter ring and PiKo mouthpiece
- 2. Adapter fitted to PiKo mouthpiece
- 3. Fit the one-way mouthpiece with a gentle twist whilst holding the adapter ring

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